

Advazorb Silflo®

Hydrophilic foam dressing with soft silicone wound contact layer and border

Product description

Advazorb Silflo® and Advazorb Silflo Lite® are absorbent, atraumatic, self-adhesive bordered dressings with a unique perforated soft silicone wound contact layer.

The silicone wound contact layer has large pores which enable the passage of exudate into the absorbent foam whilst protecting the fragile wound bed. This combination ensures the dressings comfortably stay in place whilst minimising the potential for maceration.

Use

Remove clear liners, apply pink side up to the wound ensuring the central foam pad covers the entire wound area with a minimum overlap of 2cm around the edges of the wound.



Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Advazorb Silflo® ordering information

Size	Stock code	Pack size	NHS code	Welsh Product Health Code	PIP code
7.5cm x 7.5cm	CR4190	10	-	-	-
10cm x 10cm	CR4191	10	-	-	-
12.5cm x 12.5cm	CR4192	10	-	-	-
15cm x 15cm	CR4193	10	-	-	-
10cm x 20cm	CR4194	10	-	-	-
20cm x 20cm	CR4195	10	-	-	-
10cm x 30cm	CR4196	10	-	-	-



TOP TIP

Advazorb Silflo® features a low friction backing which reduces snagging on clothing and bedsheets, prolonging dressing wear time.



Features

- ✓ Large open pore soft silicone wound contact layer
- ✓ The silicone border prevents the need for a secondary dressing or fixation
- ✓ Silicone adhesive combines security with pain free dressing changes
- ✓ Waterproof backing enables the patient to shower without the need to replace or cover the dressing

Indications

Suitable for acute and chronic exuding wounds including:

- ✓ Pre-tibial lacerations
- ✓ Cuts and abrasions
- ✓ Superficial burns
- ✓ Surgical wounds
- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers

Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilize
	Sterilized using ethylene oxide		Read instructions for use

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

