

Activon Tulle®

Knitted viscose mesh impregnated with 100% Manuka honey


Product description

Activon Tulle® is a knitted viscose mesh primary dressing impregnated with Manuka honey. Activon Tulle® creates a moist healing environment and effectively eliminates wound odour whilst providing antibacterial action.

Activon Tulle® is ideally selected for granulating or shallow wounds, it's a good choice when debriding or de-sloughing small areas of necrotic or sloughy tissue.

Use

Activon Tulle® is placed (either side down) onto the wound surface; dressings can be placed side by side to cover large wound areas or cut to size. Activon Tulle® can be unfolded to cover larger areas, this reduces the concentration of honey at the wound site. Depending on the tissue type within the wound bed and level of exudate, your secondary dressing of choice could be a film or absorbent dressing. In wounds with a high level of exudate an additional highly absorbent dressing can be introduced to help manage exudate. We recommend the Eclypse® super absorbent dressing.

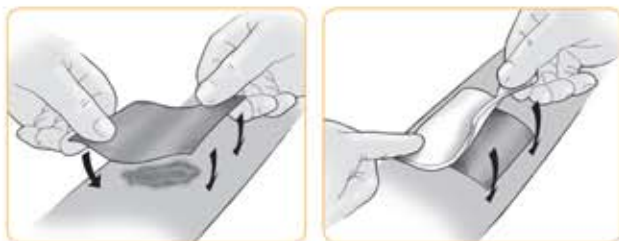

Contra-indications

Honey is not absorbed into the blood stream, however, we advise monitoring the levels of patients with diabetes. Do not use if allergic to bee venom. Discomfort can occasionally be experienced when honey is initially applied, it may be necessary to consider an appropriate level of analgesia. If discomfort continues, discontinue use and irrigate the wound with saline solution.

Activon Tulle® ordering information

Size	Stock code	Pack size	NHS code	PIP code
5cm x 5cm	CR3761	5	EJE027	304-0631
10cm x 10cm	CR3658	5	EJE028	304-0623

TOP TIP

Activon Tulle® is a triple layer knitted viscose net dressing which can be unfolded to decrease the amount of honey if discomfort is being experienced by the patient.

Algivon®, Algivon Plus® and Algivon Plus Ribbon® release honey at a slower rate into the wound therefore may be a solution if discomfort is being experienced by the patient when using Activon Tulle®.



Features

- Antibacterial
- Osmotic effect
- Enhanced healing
- Odour reduction
- Conformable
- Non-adherent

Indications

Activon Tulle® may be applied to any wound but especially sloughy, necrotic and malodorous wounds, including:

- ✓ Leg ulcers
- ✓ Diabetic ulcers
- ✓ Pressure sores
- ✓ Malodorous wounds
- ✓ Dry, sloughy or necrotic wounds
- ✓ Infected wounds

Storage

Store at room temperature. Due to the nature of honey, it can solidify at cold temperatures become more liquid at warm temperatures. If the product has hardened, warm between hands to soften before use. If the product has become too liquid, place in a colder place such as a fridge for a few minutes.

Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilise
	Sterilised using radiation		Read instructions for use

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

