

Eclipse®

Super absorbent secondary dressing


Product description

Eclipse® is a highly absorbent, high capacity wound exudate management product designed to absorb fluid, reduce potential leaks and reduce risk of maceration.

The dressing has a rapid wicking face combined with the highly absorbent moisture locking system. The absorbent layer provides a large capacity with rapid fluid uptake. The backing is a water resistant barrier film to prevent strike-through with a high moisture vapour transfer rate prolonging wear ability.

Use

Eclipse® is placed white face down on wound surface with beige backing uppermost. For large wounds several dressings can be placed side-by-side and secured with an appropriate tape or bandage. Wear time will depend on the level of exudate but Eclipse® can be left in place for up to seven days. Eclipse® can be used under compression therapy.

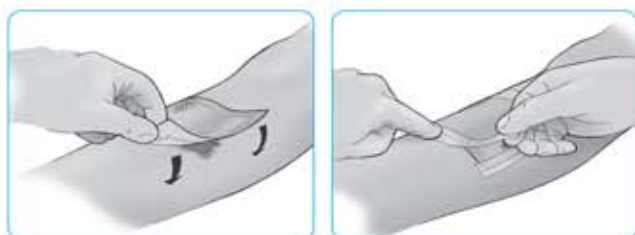

Contra-indications

Arterial bleeds and heavily bleeding wounds. Do not cut Eclipse®.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Eclipse®

Size	Stock code	Pack size	NHS code	PIP code
10cm x 10cm	CR3818	20	EJE029	372-1057
15cm x 15cm	CR3769	20	EJE032	305-1349
20cm x 30cm	CR3743	20	EJE030	306-1272
60cm x 40cm	CR3808	10	EJE034	232-1131

TOPTIP

Eclipse® has very similar technology to nappies, the dressing will expand with exudate, that's a great sign the dressing needs to be changed - the patient may notice it's become a little heavy.



Features

- ✓ Rapid absorption
- ✓ High capacity
- ✓ Fluid repellent backing
- ✓ Strength and durability
- ✓ Stay dry technology
- ✓ Thin and conforming

Indications

Moderate to heavily exuding wounds including:

- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Sloughy wounds
- ✓ Granulating wounds
- ✓ Post operative sutured or dehisced wounds
- ✓ Fungating wounds
- ✓ Donor site management

Care and use symbols

	Gaution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilize
	Sterilized using ethylene oxide		Read instructions for use

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

