

Silflex®

Soft silicone wound contact layer


Product description

A wound contact layer made from polyester mesh coated with Silfix® soft silicone.

Silflex® is designed to gently adhere to the skin surrounding a wound and not to the wound bed this minimises the pain and trauma associated with dressing change. The dressing allows the passage of exudate.

Use

Remove the clear liners on each side of the dressing and place directly over the wound. Silflex® may overlap the wound edges. The dressing can be cut to size ensuring that sharp scissors are used. Cover with a secondary dressing of choice which may be an absorbent pad or film dressing. Silflex® may be kept in place for up to fourteen days depending on the wound condition.

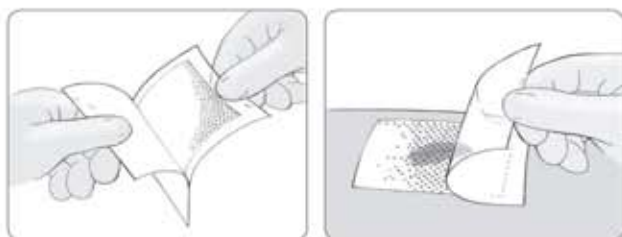

Contra-indications

Do not use if allergic to silicone.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Silflex®

Size	Stock code	Pack size	NHS code	PIP code
5cm x 7cm	CR3922	10	EKH028	338-4385
8cm x 10cm	CR3923	10	EKH029	338-4393
12cm x 15cm	CR3924	10	EKH030	338-4401
20cm x 30cm	CR3925	10	EKH032	338-4419
35cm x 60cm	CR4006	5	EKH042	348-2205



Features

- ✓ Adheres only to dry skin not to a moist wound
- ✓ Atraumatic soft silicone, reduced pain on dressing change
- ✓ Allows passage of exudate
- ✓ Hydrophobic Silfix® soft silicone coating
- ✓ Soft and conformable

Indications

- ✓ Skin tears
- ✓ Skin abrasions
- ✓ Surgical wounds
- ✓ Second degree burns
- ✓ Lacerations
- ✓ Leg and pressure ulcers
- ✓ Topical negative pressure (primary dressing)

Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilize
	Sterilized using ethylene oxide		Read instructions for use



TOPTIP

Wet your hands/gloves before application so that the Silflex® doesn't stick when applying.

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

