

Advadraw®

Rapid capillary action dressing

Product description

An absorbent, non-adherent dressing with a rapid wicking effect. Advadraw® has a unique integrated central wicking layer between absorbent polyester/ viscose fibres lined with perforated non-adherent film. Fluid from the wound bed is rapidly absorbed into the dressing and distributed by the central layer resulting in sustained movement of exudate from a wound bed.

The wicking of fluid from the wound bed will promote autolytic debridement, deslough and debride wounds and help remove bacteria in the exudate.

Use

Advadraw® is placed (either side down) onto the wound surface. Dressings can be placed side by side to cover large wound areas and layered to optimise absorbency. Advadraw® can be cut to size ensuring that sharp scissors are used. Secure with a secondary dressing of choice.

Depending on wound exudate levels, any surrounding interstitial fluid, oedema and number of layers used, Advadraw® may initially require changing daily but can be extended and left in place for up to 7 days. Advadraw® will need changing when the top up layer becomes saturated to the dressing margins.



Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage


Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Advadraw®

Size	Stock code	Pack size	NHS Code	PIP code
5cm x 7.5cm	CR3746	10	ELY174	304-3759
10cm x 10cm	CR3747	10	ELY175	304-3783
10cm x 15cm	CR3748	10	ELY176	304-3809
15cm x 20cm	CR3822	20	ELY287	329-5961

TOP TIP

Advadraw® can be stacked in layers on top of each other. If you feel there is high exudate you can remove top layers and apply new ones. This helps reduce unnecessary dressing changes and reducing trauma to wound and infection.

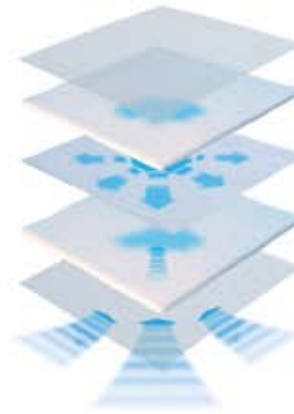

Features

- ✓ Rapid absorption
- ✓ Draws in exudate
- ✓ Capillary action
- ✓ Non-adherent
- ✓ Easy to cut
- ✓ Soft and conformable
- ✓ Highly absorbent

Indications

All acute and chronic wounds, especially sloughy, necrotic or exuding wounds including:

- ✓ Leg ulcers
- ✓ Post operative or dehisced surgical wounds
- ✓ Pressure ulcers

Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilize
	Sterilised using ethylene oxide		Read instructions for use

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

