

Advadraw Spiral®

Rapid capillary action ribbon



Product description

An absorbent, low adherent dressing with a wicking effect, presented in a spiral ribbon form. The dressing comprises of a soft viscose and polyester fibres with a central wicking layer with a perforated permeable wound contact layer. The unique spiral shape allows the dressing to be easily cut to length for insertion into cavity wounds or conform to deep wound beds.

Fluid from the wound bed is rapidly absorbed into the ribbon and distributed by the central layer, resulting in sustained movement of exudate from a wound bed. This helps to manage exudate levels and promote autolytic debridement.

Use

Advadraw Spiral® can be inserted into a wound or placed (either side down) onto the wound surface. The end of Advadraw Spiral® can be inserted into a standard Advadraw® dressing which will act as a reservoir for exudate collection. Advadraw Spiral® can be cut to length. Secure with a secondary dressing e.g. a film dressing, bandage and/or tape. Advadraw Spiral® may initially require changing daily but can be left in place for up to seven days.



Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Ordering information for Advadraw Spiral®

Size	Stock code	Pack size	NHS code	PIP code
0.5cm x 40cm	CR3799	10	ELY198	314-8640

Features

- ✓ Rapid absorption
- ✓ Draws in exudate
- ✓ Capillary action
- ✓ Can be used directly on wound bed

Indications

All acute and chronic wounds, especially sloughy, necrotic or exuding wounds including:

- ✓ Leg ulcers
- ✓ Post operative or dehisced surgical wounds
- ✓ Pressure ulcers
- ✓ Cavity wounds

Care and use symbols

	Caution - pay specific attention to any warnings on the IFU		CE marked medical device
	Latex free		Keep dry
	Single use only		Store out of direct sunlight
	Do not use if packaging is broken or damaged		Do not re-sterilize
	Sterilised using ethylene oxide		Read instructions for use

TOPTIP

Advadraw Spiral® doesn't go soft in the cavity unlike hydrofibres do. Less chance of losing the dressing in the wound and less complications of removing the dressing.

Advancis
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

