

Advazorb Silfix®

Hydrophilic foam dressing with soft silicone wound contact layer


Product description

Advazorb Silfix® and Advazorb Silfix Lite® combine the excellent fluid handling capabilities of Advazorb® with a layer of atraumatic soft silicone adhesive.

The soft silicone ensures that the dressing does not stick to the wet surface of a wound bed or cause trauma to new tissue upon dressing removal. This is particularly suited to those patients with friable or delicate skin and an ideal choice for sensitive wounds.

Use

Remove clear liners and apply pink side up to the wound ensuring the dressing covers the entire wound area and a minimum overlap of 2cm around the edges of the wound. Secure in place with tape, appropriate bandage or secondary dressing.

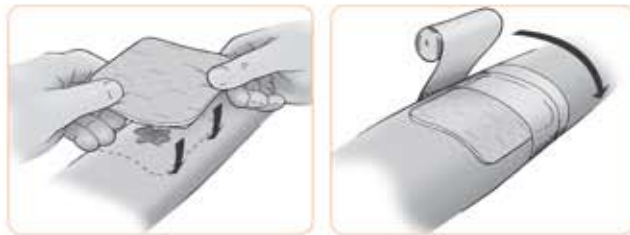

Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

Storage

Store at room temperature, out of direct sunlight and keep dry.

Advazorb Silfix® ordering information

Size	Stock code	Pack size	NHS code	PIP code
7.5cm x 7.5cm	CR4177	10	ELA535	367-0494
10cm x 10cm	CR4178	10	ELA536	367-0502
12.5cm x 12.5cm	CR4179	10	ELA537	367-0510
15cm x 15cm	CR4180	10	ELA538	367-0528
10cm x 10cm	CR4181	10	ELA539	367-0536
20cm x 20cm	CR4182	10	ELA540	367-0544

TOP TIP

The silicone layer on Advazorb Silfix® enables the dressing to be placed onto the patient whilst a secondary dressing or bandage is applied for retention. This allows the dressing to be applied by a single health care professional.



Features

- ✓ **Atraumatic soft silicone wound contact layer**
- ✓ **Hydrophobic silicone layer will not attach to a wet wound bed**
- ✓ **Repeated dressing applications will not cause epithelial skin stripping**
- ✓ **Enables the dressing to be placed onto the patient whilst a secondary dressing or bandage is applied for retention.**

Indications

Suitable for acute and chronic exuding wounds including:

- ✓ **Cuts and abrasions**
- ✓ **Superficial burns**
- ✓ **Surgical wounds**
- ✓ **Leg ulcers**
- ✓ **Pressure ulcers**
- ✓ **Diabetic ulcers**

Care and use symbols



Adv *ancis*
Medical
www.advancis.co.uk

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

