

# Advazorb Border® (Silflo®)

Hydrophilic foam dressing with soft silicone wound contact layer and border


## Product description

Advazorb Border® and Advazorb Border Lite® are absorbent, atraumatic, self-adhesive bordered dressings with a unique perforated soft silicone wound contact layer.

The silicone wound contact layer has large pores which enable the passage of exudate into the absorbent foam whilst protecting the fragile wound bed. This combination ensures the dressings comfortably stay in place whilst minimising the potential for maceration.

## Use

Remove clear liners, apply pink side up to the wound ensuring the central foam pad covers the entire wound area with a minimum overlap of 2cm around the edges of the wound.

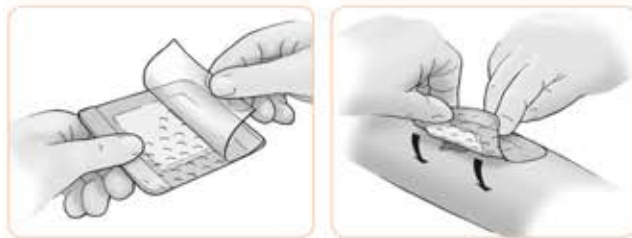

## Contra-indications

Arterial bleeds, heavily bleeding wounds and vascular fungating tumours.

## Storage

Store at room temperature, out of direct sunlight and keep dry.

## Advazorb Border® ordering information

Size	Stock code	Pack size	NHS code	PIP code
7.5cm x 7.5cm	CR4190	10	ELA547	367-0650
10cm x 10cm	CR4191	10	ELA548	367-0668
12.5cm x 12.5cm	CR4192	10	ELA549	367-0676
15cm x 15cm	CR4193	10	ELA550	367-0684
10cm x 10cm	CR4194	10	ELA551	367-0692
20cm x 20cm	CR4195	10	ELA552	367-0718
10cm x 30cm	CR4196	10	ELA553	367-0726

\* formerly Advazorb Silflo

## TOP TIP

Advazorb Border® features a low friction backing which reduces snagging on clothing and bedsheets, prolonging dressing wear time.



## Features

- ✓ Large open pore soft silicone wound contact layer
- ✓ The silicone border prevents the need for a secondary dressing or fixation
- ✓ Silicone adhesive combines security with pain free dressing changes
- ✓ Waterproof backing enables the patient to shower without the need to replace or cover the dressing

## Indications

Suitable for acute and chronic exuding wounds including:

- ✓ Pre-tibial lacerations
- ✓ Cuts and abrasions
- ✓ Superficial burns
- ✓ Surgical wounds
- ✓ Leg ulcers
- ✓ Pressure ulcers
- ✓ Diabetic ulcers

## Care and use symbols



**Advancis**  
Medical  
[www.advancis.co.uk](http://www.advancis.co.uk)

Another quality product selected and stocked by:

WM Bamford & Co Ltd p 0800-BAMFORD f 04-569-6489  
e enquiries@bamford.co.nz www.bamford.co.nz

When performance counts **BAMFORD** delivers

