



TITEMS Holder

TITEMS
TIME • TEMP • STEAM



features : instrument protection

Holder

functionality

Titems Holders are 100% Australian made products designed for ease and variety of use. This versatile holder can hold a wide range of instruments, protecting them from damage during sterilization and whilst in storage after sterilization. The holder has a clear panel enabling easy identification of instrument tips. The durable design and thick paper construction protects against puncture of sterilization bags and pouches, as well as minimizing the possibility of linting. The convenience of the holders size means that not only does it fit into a wide variety of sterilization bags and pouches, but also protects a wide range of instruments as well.

components

The Titems Holder product line includes :

Qty	Description	Code
100/Box	TITEMS Holders	AUS267

availability

Available in 5cm x 12.7cm (Cat # AUS267) for small surgical instruments up to 12.7cm in length.

implementation

Austmel Titems Holders enable the user to choose from several options enabling a snug fit of instruments increasing the prospect of successful, undamaged and unlabeled sterilization whilst enabling easy identification of the instrument.


summary

The Titems Holder

- Holds delicate surgical instruments snugly to protect them from damage during sterilization.
- Protective storage after sterilization.
- Clear viewing of instrument tip for easy identification.
- Durable, thick construction minimises the possibility of linting.
- Design protects against puncture of sterilization bag or pouch.
- Convenient size to fit a variety of instruments.
- Suitable for many sterilization bags and pouches.
- Easy to use

quick reference

Titems Holders can be used under the following conditions

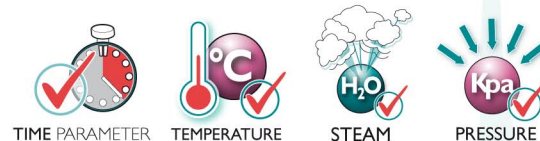

related AUSTMEL products :

- Titems Monitors • Prion Indicators
- Steam Chart 121 Integrator
- Greencard T Air Removal & Steam Penetration Test
- Titems Diagnostic Emulator • MEDITRAX® Surgery Integrator Labels
- Steri-Pak Bowie-Dick Test Pack • EZ Test Biological Indicators
- Green Card Bowie-Dick Test Indicator Card





TITEMS Holder

TITEMS
TIME • TEMP • STEAM

Holder



one-on-one Titem Diagnostic Emulator **customer support**
Austmel Pty Ltd is able to offer one-on-one support during office hours on our toll free phone service. This is a valuable service to healthcare personnel who require immediate technical support to use the Titem Diagnostic Emulator. This service is a complete one-on-one person to person level.

background

Austmel Pty Ltd is an Australian owned company established in 1974. Austmel have been directly involved in the supply and development of Sterility Assurance products for over 30 years. Our unique products and systems extend into both the Health Care and Industrials markets, including :

- MEDITRAX® Accountability & Traceability Systems
- Biological and Chemical Sterilization Indicators
- Bowie-Dick Tests
- Sterilization Packaging
- Quality Assurance Consultation
- TMI Electronic Data Loggers
- Surgical Disposables

T 1800 074 085 • NZ 0800 400 151 **customer support**



Instructions for use of the Titems Holder:

1. Insert instrument through the slots of the protector until the tip is completely covered by the plastic flap.
2. Choose the slots or holes that will best ensure a snug fit.
3. Slide the loaded instrument protector into a sterilization pouch, with instrument facing the film side.
4. Seal pouch in normal manner and sterilize.
5. Do not forget that Holder provides protection against damage in storage after sterilization.

quick reference

Titems Holders can be used under the following conditions



related AUSTMEL products :

- Titems Monitors • Prion Indicators
- Steam Chart I2I Integrator
- Greencard T Air Removal & Steam Penetration Test
- Titems Diagnostic Emulator • MEDITRAX® Surgery Integrator Labels
- Steri-Pak Bowie-Dick Test Pack • EZ Test Biological Indicators
- Green Card Bowie-Dick Test Indicator Card

