

Established 1974

Austmel

assuring your future

WASH-CHECKS™ U

Disposable Ultrasonic Cleaning Monitor

Reorder Code: WC108

- **CHECK YOUR ULTRASONIC CLEANER EFFECTIVENESS WITH A CONSISTENT MONITOR ON A ROUTINE BASIS**

When cleaning conditions are effective, the test soil on monitor is dissolved.

The presence of a large amount of blue test soil remaining on the device means a serious deficiency in ultrasonic cleaning efficiency.

- **LOW COST MONITOR PROMOTES ROUTINE USE**

We recommend monitoring the ultrasonic cleaner once daily with an empty load for a machine release and periodically throughout the day with instruments for a load release.

- **USE WITH WASH-CHECKS HOLDER**

Save money by using the same holder for your Wash-Checks Washer-Disinfector monitors and your Wash-Checks U - Ultrasonic monitors.

Wash-Checks U have a special edge that allows you to insert the monitor only half way.

BEFORE CLEANING



INSUFFICIENT CLEANING



SUFFICIENT CLEANING



HOLDER



ORDERING INFORMATION

PART #	PACKAGING
WC108	50/Bag
WC102 (Holder)	Each

T 1800 074 085 Customer Support

