



Clinitemp

Instructions for Use

Product Description: L314 Latex Free Forehead Moving Line Temperature Monitor

Standard Range: 84-106°F / 29-41°C

Indication: Monitor trends and / or measure human forehead surface temperature

Use: Ideal for use in Pre-op, Surgery, PACU or wherever temperature trend monitoring is indicated

Forehead Thermometer readings are different from; and will not give the same reading as an oral or rectal thermometer. Forehead temperature is estimated to be 3.5°F lower than core body temperature.

Clinitemp has an offset of 3.5 °F that compensates for this difference by increasing the displayed temperature by 3.5 °F. This offset can differ person to person and therefore the temperature displayed on the strip could vary from actual core temperature.

Unusual readings should be verified with an oral or rectal thermometer

Directions: Remove backing. Apply to center of forehead. Skin surface should be clean and dry.

Read temperature when yellow line stops moving.

Reversible analog line moves along the scale to show temperature readings in °F and °C.

Operating conditions: 60 to 104°F

Storage: 41 to 149°F